

WATERPROOF SKIN CARE CREAM REMOIS BARRIER



Moisturizing barrier protects the skin by waterproof function.

REMOIS BARRIER Skin-moisturizing waterproof cream.

Waterproofing and **Moisturizing** have effect of maintaining a healthy skin.

Keeps the skin clean and safe from irritation

A thin waterproof layer keeps the skin clean and protects from constant external irritation.

Worries about serious irritation caused by faeces


PH-buffering ability softly protects the skin from overstimulation. Added moisturizer keeps the skin soft.

An uncomfortable, sticky feeling

Much less of a sticky feeling on the skin. A soft and natural feeling after use.


Incontinence Area at risk of constant irritation

Mildly acidic REMOIS BARRIER, with its **pH-buffering ability**, can protect the skin by preventing the acidic irritation caused by faeces.

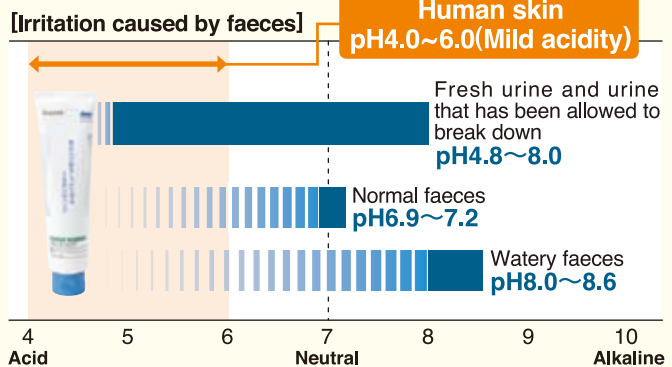


Highly waterproof quality with a less sticky feeling.

Instruction for REMOIS BARRIER

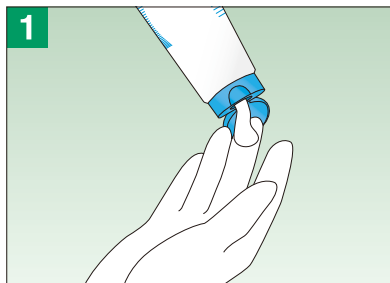

1
Take an appropriate amount (2 g for a 10 cm x 10 cm) on the fingertip.

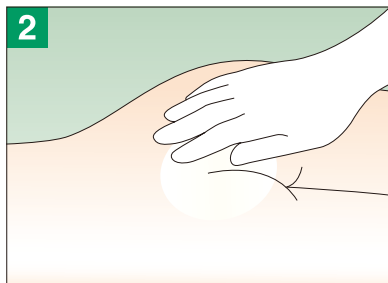

2
Apply to the skin and spread until the milky white cream turns colorless.



Its highly waterproof surface coating protects the skin from staining or irritation.

Precautions

- Do not apply the cream to broken skin or skin that has eczema or dermatitis.
- If abnormalities, reddening, irritation, etc., are noticed during or after application, immediately stop using the cream and contact a medical specialist. These symptoms may become worse if it is continually used.
- Do not apply the cream to areas of skin to which you are about to apply adhesive, such as tape, since it may easily peel off.
- If the cream is accidentally swallowed, immediately contact a medical specialist.
- Excessive friction and frequent bed-baths may weaken the cream's protective effect. Reapply as needed.
- Avoid direct sunlight and excessive heat during storage.
- Keep away from infants and children.




REMOIS BARRIER

Type	Product Code No.	Specification	Contents Per Box
Regular	18031	160g	1 pc
Mini	18032	50g	1 pc



What's REMOIS?

ALCARE proposes a new concept for skin care, "Skin Care" + "Moisture"
Moisture is added on procedure required for skin care and through the improvement of the barrier function, it will keep the skin condition healthy.

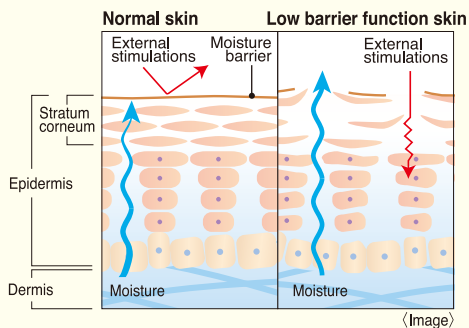

The skin with the lower barrier function will dry out due to the damaged stratum corneum, and so it will be easy to be exposed to the external stimulations by allergen* or microorganisms. To keep the skin condition healthy, moisturizing the stratum corneum is significant.

*Allergen:
It causes allergy reaction.
There are many allergen, dead mites, formaldehyde, etc.



ALCARE Co., Ltd.

International Sales Unit
19F, Arcacentral Bldg., 1-2-1 Kinshi, Sumida-ku, Tokyo 130-0013, Japan
TEL.+81-3-5611-7819 FAX.+81-3-5611-7827 MAIL. info@alcare.co.jp
www.alcare.co.jp/e